

December 21, 2020

Super Liu  
Hangzhou Biotest Biotech Co., Ltd.  
No.17 Futai Road, Zhongtai Street, Yuhang District  
Hangzhou, Zhejiang, P.R.China, 311121

Device: RightSign COVID-19 IgG/IgM Rapid Test Cassette

Company: Hangzhou Biotest Biotech Co., Ltd.

Indication: This test is indicated for the following indications for use:

For certain authorized laboratories (see below) - Qualitative detection and differentiation of IgM and IgG antibodies to SARS-CoV-2 in human venous whole blood (sodium heparin, potassium EDTA and sodium citrate), serum or plasma (sodium heparin, potassium EDTA and sodium citrate). Intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection.

For certain authorized laboratories (see below) - Qualitative detection and differentiation of IgM and IgG antibodies to SARS-CoV-2 in fingerstick whole blood. Intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection.

Authorized Laboratories: Testing of serum, plasma and venous whole blood specimens is limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. 263a, that meet requirements to perform moderate or high complexity tests.

Testing of fingerstick whole blood specimens is limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, that meet the requirements to perform high, moderate or waived complexity tests. Testing of fingerstick whole blood specimens is authorized for use at the Point of Care (POC), i.e., in patient care settings operating under a CLIA Certificate of Waiver, Certificate of Compliance, or Certificate of Accreditation.

Dear Mr. Liu:

On June 4, 2020, based on your<sup>1</sup> request, the Food and Drug Administration (FDA) issued an Emergency Use Authorization (EUA) for emergency use of your product,<sup>2</sup> for the qualitative detection and differentiation of IgM and IgG antibodies against SARS-CoV-2 in human serum, plasma, and venous whole blood (sodium heparin, potassium EDTA, and sodium citrate), as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection pursuant to Section 564 of the Federal Food, Drug, and Cosmetic Act (the Act) (21 U.S.C. §360bbb-3). Testing was limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, to perform moderate or high complexity tests. Subsequently, on July 23, 2020,<sup>3</sup> and again on August 31, 2020,<sup>4</sup> FDA granted your request to update the authorized labeling.

On September 15, 2020, you requested to revise your EUA. Based on that request, and having concluded that revising the June 4, 2020, EUA is appropriate to protect the public health or safety under section 564(g)(2)(C) of the Act (21 U.S.C. § 360bbb-3(g)(2)(C)), FDA is reissuing the June 4, 2020, letter in its entirety with the revisions incorporated.<sup>5</sup> Pursuant to section 564 of the Act and the Scope of Authorization (Section II) and Conditions of Authorization (Section IV) of this reissued letter, your product is now authorized for use consistent with the indication described above.

On February 4, 2020, pursuant to Section 564(b)(1)(C) of the Act, the Secretary of the Department of Health and Human Services (HHS) determined that there is a public health emergency that has a significant potential to affect national security or the health and security of United States citizens living abroad, and that involves the virus that causes COVID-19. Pursuant to Section 564 of the Act, and on the basis of such determination, the Secretary of HHS then declared that circumstances exist justifying the authorization of emergency use of in

---

<sup>1</sup> For ease of reference, this letter will use the term “you” and related terms to refer to Hangzhou Biotest Biotech Co., Ltd.

<sup>2</sup> For ease of reference, this letter will use the term “your product” to refer to the RightSign COVID-19 IgG/IgM Rapid Test Cassette. Premier Biotech Inc.’s brand name for your product is Premier Biotech COVID-19/IgG/IgM Rapid Test Cassette. CLIAWaived Inc.’s brand name for your product is CoronaCHEK COVID-19 IgG/IgM Rapid Test Cassette.

<sup>3</sup> On July 23, 2020, your request was granted to update the Instructions for Use (IFU) of your product to include the use of external positive and negative controls and to add four additional authorized distributors, two with additional brand names.

<sup>4</sup> On August 31, 2020, your request was granted to (1) update the IFU to include the RightSign “SARS-COV-2 IgG/IgM External Control Kit” as a material required but not provided; (2) update the authorized labeling to include the “Instruction for use of RightSign SARS-COV-2 IgG/IgM External Control Kit,” and (3) update the authorized labeling for the two authorized distributors who will distribute the RightSign COVID-19 IgG/IgM Rapid Test Cassette and the RightSign SARS-COV-2 IgG/IgM External Control Kit under their own brand names.

<sup>5</sup> The revisions to the June 4, 2020, letter include: (1) addition of fingerstick whole blood as an authorized specimen type for laboratories certified under CLIA that meet requirements to perform high, moderate or waived complexity tests, (2) conforming updates to the authorized labeling, (3) clarifying revisions to the indication and Conditions of Authorization, and (4) addition of two quick reference guides for use with fingerstick specimens, “RightSign COVID-19 IgG/IgM Rapid Test Cassette Format 1,” and “RightSign COVID-19 IgG/IgM Rapid Test Cassette Format 2.”

vitro diagnostics for detection and/or diagnosis of the virus that causes COVID-19 subject to the terms of any authorization issued under Section 564(a) of the Act.<sup>6</sup>

FDA considered the totality of scientific information available in authorizing the emergency use of your product for the indication above. A summary of the performance information FDA relied upon is included in the “RightSign COVID-19 IgG/IgM Rapid Test Cassette Package Insert” Instructions for Use.

Having concluded that the criteria for issuance of this authorization under Section 564(c) of the Act are met, I am authorizing the emergency use of your product, described in the Scope of Authorization of this letter (Section II), subject to the terms of this authorization.

### **I. Criteria for Issuance of Authorization**

I have concluded that the emergency use of your product meets the criteria for issuance of an authorization under Section 564(c) of the Act, because I have concluded that:

1. The SARS-CoV-2 can cause a serious or life-threatening disease or condition, including severe respiratory illness, to humans infected by this virus;
2. Based on the totality of scientific evidence available to FDA, it is reasonable to believe that your product may be effective in diagnosing recent or prior infection with SARS-CoV-2 by identifying individuals with an adaptive immune response to the virus that causes COVID-19, and that the known and potential benefits of your product when used for such use, outweigh the known and potential risks of your product; and
3. There is no adequate, approved, and available alternative to the emergency use of your product.<sup>7</sup>

### **II. Scope of Authorization**

I have concluded, pursuant to Section 564(d)(1) of the Act, that the scope of this authorization is limited to the indication above.

### **Authorized Product Details**

Your product is a qualitative test for the detection and differentiation of IgM and IgG antibodies to SARS-CoV-2 in human venous whole blood (sodium heparin, potassium EDTA and sodium citrate), serum or plasma (sodium heparin, potassium EDTA and sodium citrate), and fingerstick whole blood. Your product is intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection.

---

<sup>6</sup> U.S. Department of Health and Human Services, *Determination of a Public Health Emergency and Declaration that Circumstances Exist Justifying Authorizations Pursuant to Section 564(b) of the Federal Food, Drug, and Cosmetic Act*, 21 U.S.C. § 360bbb-3. 85 FR 7316 (February 7, 2020).

<sup>7</sup> No other criteria of issuance have been prescribed by regulation under Section 564(c)(4) of the Act.

Testing of serum, plasma and venous whole blood specimens is limited to laboratories certified under CLIA, that meet requirements to perform moderate or high complexity tests.

Testing of fingerstick whole blood specimens is limited to laboratories certified under the CLIA, that meet the requirements to perform high, moderate or waived complexity tests. Testing of fingerstick whole blood specimens is authorized for use at the POC, i.e., in patient care settings operating under a CLIA Certificate of Waiver, Certificate of Compliance, or Certificate of Accreditation.

At this time, it is unknown for how long antibodies persist following infection and if the presence of antibodies confers protective immunity.

To use your product, the device cassette, specimen, and sample buffer are allowed to equilibrate to room temperature. Specimen (10 µL) is transferred to the specimen well of the cassette. Then, 2 drops of buffer are added to the buffer well. Wait for 10 minutes and read the test results. Results are not to be read after 20 minutes. An IgM Positive Result occurs when a colored line appears at the IgM test line region and a colored line appears at the control line region; this indicates that IgM against SARS-CoV-2 is present. An IgG Positive Result occurs when a colored line appears at the G test line region and a colored line appears at the control line region; this indicates that IgG against SARS-CoV-2 is present. A positive result for IgM and IgG occurs when colored lines occur at both IgM and IgG test line regions as well as a colored line at the control line region. A Negative Result occurs when the colored line appears at the control line region only and indicates that IgM and IgG antibodies against SARS-CoV-2 were not detected. An Invalid Result occurs when the colored line at the control line region fails to appear; the test should then be repeated.

Your product requires the following internal control, that is processed along with the specimen on the device cassette. The internal control listed below must generate expected results in order for a test to be considered valid, as outlined in the “RightSign COVID-19 IgG/IgM Rapid Test Cassette Package Insert” Instructions for Use:

- Internal Control – The colored line at the control line region should appear for every test and checks that flow of reagents is satisfactory.

Your product also requires the following authorized external positive and negative controls, which are supplied separately with the “RightSign SARS-COV-2 IgG/IgM External Control Kit”<sup>8</sup> Instructions for Use, or other authorized controls (refer to Condition L below). Your product also requires the use of additional authorized materials and authorized ancillary reagents that are not included with your product and are described in the Instructions for Use.

Your above described product is authorized to be accompanied with labeling entitled “RightSign COVID-19 IgG/IgM Rapid Test Cassette Package Insert” Instructions for Use (available at <https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/vitro-diagnostics-euas>), the applicable quick reference guide (depending on buffer vial type) “RightSign COVID-19 IgG/IgM Rapid Test

---

<sup>8</sup> Additional brand names of these controls are Premier Biotech COVID-19 IgG/IgM External Control Kit and CoronaCHEK COVID-19 IgG/IgM External Control Kit.

Cassette Format 1,” or “RightSign COVID-19 IgG/IgM Rapid Test Cassette Format 2,” and the following product-specific information pertaining to the emergency use, which is required to be made available to healthcare providers and recipients (collectively referenced as “authorized labeling”):

- Fact Sheet for Healthcare Providers: RightSign COVID-19 IgG/IgM Rapid Test Cassette
- Fact Sheet for Recipients: RightSign COVID-19 IgG/IgM Rapid Test Cassette.

The above described product, when accompanied by the authorized labeling, is authorized to be distributed to and used by authorized laboratories under this EUA, despite the fact that it does not meet certain requirements otherwise required by applicable federal law.

I have concluded, pursuant to Section 564(d)(2) of the Act, that it is reasonable to believe that the known and potential benefits of your product, when used consistent with the Scope of Authorization of this letter (Section II), outweigh the known and potential risks of your product.

I have concluded, pursuant to Section 564(d)(3) of the Act, based on the totality of scientific evidence available to FDA, that it is reasonable to believe that your product may be effective in diagnosing recent or prior infection with SARS-CoV-2 by identifying individuals with an adaptive immune response to the virus that causes COVID-19, when used consistent with the Scope of Authorization of this letter (Section II), pursuant to Section 564(c)(2)(A) of the Act.

FDA has reviewed the scientific information available to FDA, including the information supporting the conclusions described in Section I above, and concludes that your product (as described in the Scope of Authorization of this letter (Section II) meets the criteria set forth in Section 564(c) of the Act concerning safety and potential effectiveness.

The emergency use of your product under this EUA must be consistent with, and may not exceed, the terms of this letter, including the Scope of Authorization (Section II) and the Conditions of Authorization (Section IV). Subject to the terms of this EUA and under the circumstances set forth in the Secretary of HHS's determination under Section 564(b)(1)(C) of the Act described above and the Secretary of HHS's corresponding declaration under Section 564(b)(1) of the Act, your product is authorized for the indication above.

### **III. Waiver of Certain Requirements**

I am waiving the following requirements for your product during the duration of this EUA:

- Current good manufacturing practice requirements, including the quality system requirements under 21 CFR Part 820 with respect to the design, manufacture, packaging, labeling, storage, and distribution of your product but excluding Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, 21 CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).

### **IV. Conditions of Authorization**

Pursuant to Section 564(e) of the Act, I am establishing the following conditions on this authorization:

**Hangzhou Biotest Biotech Co., Ltd. (You) and Authorized Distributor(s)<sup>9</sup>**

- A. Your product must comply with the following labeling requirements under FDA regulations: the intended use statement (21 CFR 809.10(a)(2), (b)(2)); adequate directions for use (21 U.S.C. 352(f)), (21 CFR 809.10(b)(5), (7), and (8)); appropriate limitations on the use of the device including information required under 21 CFR 809.10(a)(4); and any available information regarding performance of the device, including requirements under 21 CFR 809.10(b)(12).
- B. You and authorized distributor(s) must make your product available with the authorized labeling to authorized laboratories.
- C. You and authorized distributor(s) must make available on your website(s) the Fact Sheet for Healthcare Providers and the Fact Sheet for Recipients.
- D. You and authorized distributor(s) must inform authorized laboratories and relevant public health authorities of this EUA, including the terms and conditions herein, and any updates made to your product and authorized labeling.
- E. Through a process of inventory control, you and authorized distributor(s) must maintain records of the authorized laboratories to which they distribute the test and number of tests they distribute.
- F. You and authorized distributor(s) must collect information on the performance of your product. You must report to FDA any suspected occurrence of false positive and false negative results and significant deviations from the established performance characteristics of the product of which you become aware.
- G. You and authorized distributor(s) are authorized to make available additional information relating to the emergency use of your product that is consistent with, and does not exceed, the terms of this letter of authorization.
- H. You and authorized distributor(s) must make available the control material (RightSign SARS-COV-2 IgG/IgM External Control Kit) with the “RightSign SARS-COV-2 IgG/IgM External Control Kit” Instructions for Use, or other authorized control materials (as may be requested under Condition L below), at the same time as your product.

**Hangzhou Biotest Biotech Co., Ltd. (You)**

---

<sup>9</sup> “Authorized Distributor(s)” are identified by you, Hangzhou Biotest Biotech Co., Ltd., in your EUA submission as an entity allowed to distribute your product.

- I. You must notify FDA of any authorized distributor(s) of your product, including the name, address, and phone number of any authorized distributor(s).
- J. You must provide authorized distributor(s) with a copy of this EUA and communicate to authorized distributor(s) any subsequent amendments that might be made to this EUA and its authorized accompanying materials (e.g., Fact Sheets).
- K. You must comply with the following requirements pursuant to FDA regulations: Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, 21 CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).
- L. You may request changes to this EUA for your product, including to the Scope of Authorization (Section II in this letter) or to the authorized labeling, including requests to make available additional authorized labeling specific to an authorized distributor. Such additional labeling may use another name for the product but otherwise must be consistent with the authorized labeling, and not exceed the terms of authorization of this letter. Any request for changes to this EUA should be submitted to the Division of Microbiology (DMD)/Office of Health Technology 7 (OHT7)-Office of In Vitro Diagnostics and Radiological Health (OIR)/Office of Product Evaluation and Quality (OPEQ)/Center for Devices and Radiological Health (CDRH) and require appropriate authorization from FDA prior to implementation.
- M. You must evaluate the performance and assess traceability<sup>10</sup> of your product with any FDA-recommended reference material(s) or established panel(s) of characterized clinical specimens. After submission to and concurrence with the data by FDA, you will update your labeling to reflect the additional testing.
- N. You must have a process in place to track adverse events, including any occurrence of false results and report to FDA pursuant to 21 CFR Part 803.
- O. You must have lot release procedures and the lot release procedures, including the study design and statistical power, must ensure that the tests released for distribution have the clinical and analytical performance claimed in the authorized labeling.
- P. If requested by FDA, you must submit lot release procedures to FDA, including sampling protocols, testing protocols, and acceptance criteria, that you use to release lots of your product for distribution in the U.S. If such lot release procedures are requested by FDA, you must provide it within 48 hours of the request.
- Q. If requested by FDA, you must periodically submit new lots for testing at the National Cancer Institute, or by another government agency designated by FDA, to confirm continued performance characteristics across lots. In addition, FDA may request records

---

<sup>10</sup> Traceability refers to tracing analytical sensitivity/reactivity back to an FDA-recommended reference material. FDA may request, for example, that you perform this study in the event that we receive reports of adverse events concerning your product.

regarding lot release data for tests to be distributed or already distributed. If such lot release data are requested by FDA, you must provide it within 48 hours of the request.

- R. You must complete the agreed upon real-time stability study for your product. After submission to and concurrence with the data by FDA, you will update your labeling to reflect the additional testing.

#### **Authorized Laboratories**

- S. Authorized laboratories using your product must include with test result reports, all authorized Fact Sheets. Under exigent circumstances, other appropriate methods for disseminating these Fact Sheets may be used, which may include mass media.
- T. Authorized laboratories must use your product as outlined in the authorized labeling. Deviations from the authorized procedures, including the authorized clinical specimen types, authorized control materials, authorized other ancillary reagents and authorized materials required to use your product are not permitted.
- U. Authorized laboratories that receive your product must notify the relevant public health authorities of their intent to run your product prior to initiating testing.
- V. Authorized laboratories using your product must have a process in place for reporting test results to healthcare providers and relevant public health authorities, as appropriate.
- W. Authorized laboratories must collect information on the performance of your product and report to DMD/OHT7-OIR/OPEQ/CDRH (via email: [CDRH-EUA-Reporting@fda.hhs.gov](mailto:CDRH-EUA-Reporting@fda.hhs.gov)) and you (info.usa@biotests.com.cn) any suspected occurrence of false positive or false negative results and significant deviations from the established performance characteristics of your product of which they become aware.
- X. All laboratory personnel using your product must be appropriately trained in immunochromatographic techniques and use appropriate laboratory and personal protective equipment when handling this kit, and use your product in accordance with the authorized labeling. All laboratory personnel using the assay must also be trained in and be familiar with the interpretation of results of the product.

#### **Hangzhou Biotest Biotech Co., Ltd. (You), Authorized Distributors and Authorized Laboratories**

- Y. You, authorized distributors, and authorized laboratories using your product must ensure that any records associated with this EUA are maintained until otherwise notified by FDA. Such records will be made available to FDA for inspection upon request.

#### **Conditions Related to Advertising and Promotion**

- Z. All descriptive printed matter, advertising, and promotional materials relating to the use



of your product, shall be consistent with authorized labeling, as well as the terms set forth in this EUA and meet the requirements set forth in section 502(a), (q)(1), and (r) of the Act and FDA implementing regulations.

AA. No descriptive printed matter, advertising, or promotional materials relating to the use of your product, may represent or suggest that this test is safe or effective for the detection of SARS-CoV-2.

BB. All descriptive printed matter, advertising, and promotional materials relating to the use of your product, shall clearly and conspicuously state that:

- This test has not been FDA cleared or approved but has been authorized by FDA under an EUA for use by authorized laboratories;
- This test has been authorized only for the presence of IgM and IgG antibodies to SARS-CoV-2, not for any other viruses or pathogens; and
- The emergency use of this test is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

The emergency use of your product as described in this letter of authorization must comply with the conditions and all other terms of this authorization.

#### **V. Duration of Authorization**

This EUA will be effective until the declaration that circumstances exist justifying the authorization of the emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 is terminated under Section 564(b)(2) of the Act or the EUA is revoked under Section 564(g) of the Act.

Sincerely,

---

RADM Denise M. Hinton  
Chief Scientist  
Food and Drug Administration

Enclosures